

TV/VCR Remote

Works like a Universal Remote with only one surface to activate. Compatible with most TV's and VCR's.



Set-up

Customize for your own television.

1. Select TV function on the remote.
2. Program in your TV code.
3. First activation turns the television "on."
4. Second through fifth activations advance the TV channels one at a time.*
5. Sixth activation turns the television "off."

* You can customize any number of channels in any order. See product manual.

Customize for your own VCR.

1. Select VCR function on the remote.
2. Program in your VCR code.
3. Insert tape in your VCR, turn VCR off.
4. First activation turns the TV and VCR "on," turns the television to the appropriate channel and plays the VCR.
5. Second activation stops the VCR and turns both the TV and VCR "off."

Go

Watch a Sporting Event

Here's what you need:

- TV/VCR Remote
- Switch (if needed)

1. Place remote in TV function.
2. Face remote towards television.
3. Activate the remote to find a sporting event.

Grow

Watch a Sporting Event and Cheer

Additional materials you'll need:

- One-Step Communicator²⁰
- Boardmaker™ Software
- Snap Switch Cap

1. Select a message for the One-Step Communicator²⁰. Examples: "GO TEAM GO!" "Yeah, Vikings!"
2. Use Boardmaker to develop a symbol for cheering.
3. Place symbol on the One-Step and secure with a Snap switch cap.
4. Use Track Link (included) to connect the One-Step and the TV/VCR Remote.
5. Follow steps 1-3 from the "Go" activity and cheer your team on!

More to know

Want more opportunities to increase participation? Try these!

- Watch a video of a past birthday
- Enjoy a music video
- Sing along with Sesame Street
- Have a Disney marathon
- Catch your favorite soap opera
- Rent your favorite movie
- Control educational videos in class
- Watch classroom presentations to critique



Tools to fuel your imagination

Visit our website at www.ablenetinc.com or call 1-800-322-0956 to order!



TV/VCR Remote

Only one surface makes it easy to control both a TV and a VCR. Programming instructions included. Individuals who need alternative access can simply plug in a switch.

Product Information:

9-volt battery required to operate. 1 year warranty.

Other Resources from AbleNet

Switches — Three sizes to choose from. See the information at right!

One-Step Communicator²⁰ — A Jelly Bean-sized BIGmack[®] with 20 seconds of message capability.

Step-by-Step Communicator⁷⁵ — 75 seconds of memory allows you to record any number of messages in sequence. Each activation plays the messages in the order recorded.

Step-by-Step Communicator⁷⁵ with Levels — 75 seconds of memory lets you record a series of message sequences. Save time by prerecording all the messages for an individual's daily activities at once!

Book of Possibilities — Successful, easy-to-use inclusion strategies for elementary and secondary-aged students.

Switches

Durable and reliable, AbleNet switches work every time, no matter what part of the surface is touched. Pick the size that best matches the abilities of the individual using it. Each provides audible feedback. Available in a variety of colors. Sturdy construction. 3 year warranty.



Big Red[®] Switch

5" target area is ideal for people with visual, cognitive or physical disabilities. 6' cord with 1/8" (3.5mm) mono-plug.



Jelly Bean[®] Switch

2-1/2" target area. 6' cord with 1/8" (3.5mm) monoplug.



Specs[®] Switch Package

1-3/8" target area. Three mounting options provided.

Partial Participation

An individual with disabilities can meaningfully participate in an activity regardless of ability level. Using assistive technology, even those with the most severe disabilities can activate a switch to turn on a hair dryer to blow dry hair or run an electric stapler to staple the PTA newsletter. All people, regardless of their physical or cognitive abilities, can play a purposeful role, be involved and feel pride and ownership.

No Prerequisites

There are no prerequisites to using assistive technology. Anyone, regardless of their ability, can be included in activities that provide opportunities for communication and switch activation before they may show understanding of a skill or concept. In other words, people learn to communicate by communicating!

Symbols

Symbols are the objects, photographs or line drawings that are used to represent messages and/or actions. Communication symbols are important to people with severe disabilities. It is beneficial to pair an appropriate symbol with each message and/or action.



Tools to fuel your imagination